

Tailback 2-64

Universal Back Brace



INSTRUCTIONS FOR USE

DESCRIPTION OF SYMBOLS
 Medical Device


ASSEMBLY/FITTING INSTRUCTIONS



(Figure 1) Match the size using the chart to fit your waist match your belts size. Remove Hook & Loop from belts and pull the tabs through to the sizing creases, securing the sizing flap with Hook & Loop to your appropriate crease 1,2,3,4,5,6,7. For your appropriate size, see green chart below and reference sizes.



Slip the brace over your head like a T-shirt. Position the brace so it is centered on your back and comfortably aligned with your spine's curve.

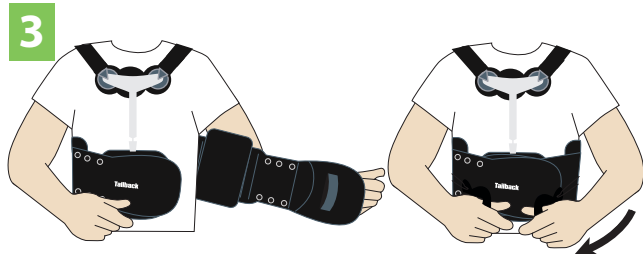

Making sure the back panel is centered, wrap the right side around to the abdomen and attach to the left panel with hook closure.

holding the back panel in one hand, use the other hand to gently pull the posterior support to the desired height. Re-attach the Hook & Loop strip to secure the support in place and re-attach the mesh backing. To adjust front metal support, first detach the two side panel of the front brace. Peel the oval foam bag and exposing all metal stay. Use the wrench to loose screws. According to your height, adjust the length and angle. And then tighten up all screws. Finally put the metal stay back into foam bag.

ICE PACK INSTRUCTIONS: THIS CAN HOOK & LOOP INTO THE BRACE



Place ice pack into the freezer for at least 4 hours before using. Place the ice pack into pouch and seal shut. Apply to affected area. Remove pack if it becomes uncomfortable. Always follow your healthcare profession's or physician's specific treatment instructions.

Caution! Please read carefully prior to product use: Supervision is advised when applying this product on geriatric patients and children. Never apply ice pack(s) directly to skin. Use of an additional protective barrier such as a cloth towel is recommended to protect against frostbite to the affected area. In the unlikely event there is exposure to the eyes, immediately flush with water and consult with a healthcare professional as a precaution. If ice pack is punctured, dispose of it and reorder a replacement to continue treatment.

CLEANING AND CARE

The Tailback may be cleaned by hand using a mild soap and warm water. Hang to dry. Do not place in washing machine or dryer.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

SYMBOLS GLOSSARY

For an explanation of symbols, please refer to www.manamed.com/glossary/symbol-glossary.pdf

TD-TB064-2-IFU Rev A

Tailback can be adjusted from 25" to 68". Tailback comes with a main panel and two adjustable belts with comfort pull rings. Simply place the Tailback on a table with the inside facing up. When using each matching number, the waist is measured as the following:

S & S	=	25" - 32" waist
M & M	=	32" - 38" waist
L & L	=	38" - 44" waist
XL & XL	=	44" - 50" waist
2XL & 2XL	=	50" - 56" waist
3XL & 3XL	=	56" - 62" waist
4XL & 4XL	=	62" - 68" waist



REQUIRED COMPONENTS
 Back brace, ice pack.

PRODUCT DESCRIPTION

The Tailback 2-64 is a universal spinal orthosis designed with our dual-pull system. It provides a rigid, comfortable, and easy-to-apply solution for every patient. Ice pack included.

INDICATIONS FOR USE

For non-surgical or postoperative support and immobilization of the injured spine.

GENERAL SAFETY INSTRUCTIONS / WARNINGS AND PRECAUTIONS

Read these instructions carefully before use. Keep them for future reference. Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities. The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of device.
- The device is for single patient, multiple use.
- This device is a supportive brace only and is not intended or guaranteed to prevent injury.
- Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.
- If you do not agree to the above conditions, contact your health care provider before using this product.

LIABILITY

ManaMed does not assume liability for the following:

- Device not maintained as instructed by the Instructions For Use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.
- ManaMed accepts no responsibility for any injury which might occur while using ManaMed products.



MANAMED

ManaMed, LLC
 2612 Sirius Dr.
 Denton, TX 76208
 Ph: 888-508-0712
www.Manamed.com

©2025 ManaMed LLC. All rights reserved.
 ManaMed, the ManaMed logo, are trademarks of ManaMed LLC
 Registered in the U.S.