

# Mini Kahuna Brace

## INSTRUCTIONS FOR USE

### Shoulder Brace with Abduction Pillow

DESCRIPTION OF SYMBOLS

**MD** Medical Device



#### REQUIRED COMPONENTS

(Figure 1)

Sling with Hook and Loop closure, shoulder strap, Abduction Pillow with waistband strap, therapy ball.

#### PRODUCT DESCRIPTION

The Mini Kahuna Brace with Abduction Pillow is universal and can be applied to the right or the left side of the patient. When properly applied, the Mini Kahuna Brace will position the arm at 10-15 degrees of abduction.

#### INDICATIONS FOR USE

For non-surgical or postoperative support and immobilization of the injured shoulder when mild abduction of the shoulder is required.

#### ASSEMBLY/FITTING INSTRUCTIONS

(Figure 2)

Open Hook & Loop strap (a) and (b), also open buckle on the arm sling buckle (c). Place arm in the brace and secure both Hook & Loop strap (a) & (b) comfortably around arm. To fit the sling to your arm, fold the sling along the pre-fold lines at the wrist to the desired length (c).

(Figure 3)

Place shoulder strap (d) over the opposite shoulder and connect buckle. Adjust the sling (e) to fit comfortably around the shoulder.

(Figure 4)

Position the abduction pillow (g) on the injured side of the body between the sling and the body. The pillow should run parallel to the forearm. Secure the pillow to the sling by pressing the hook and loop together.

(Figure 5)

Wrap the waistband (f) around the body. Engage the buckle and adjust waist band to desired fit.

#### CLEANING AND CARE

The Mini Kahuna Brace may be cleaned by hand using a mild soap and warm water. Hang to dry. Do not place in washing machine or dryer.

#### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

#### GENERAL SAFETY INSTRUCTIONS / WARNINGS AND PRECAUTIONS

Read these instructions carefully before use. Keep them for future reference. Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities. The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of device.
- The device is for single patient, multiple use.
- This device is a supportive brace only and is not intended or guaranteed to prevent injury.
- Manamed accepts no responsibility for any injury which might occur while using Manamed products.
- Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.
- Should not be used beyond 180 days.
- If you do not agree to the above conditions, contact your health care provider before using this product.





#### LIABILITY

Manamed does not assume liability for the following:

- Device not maintained as instructed by the Instructions For Use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

#### SYMBOL GLOSSARY

For an explanation of symbols, please refer to [www.manamed.com/glossary/symbol-glossary.pdf](http://www.manamed.com/glossary/symbol-glossary.pdf)

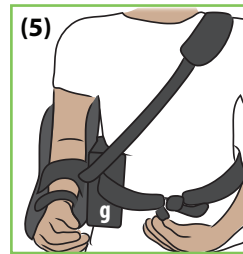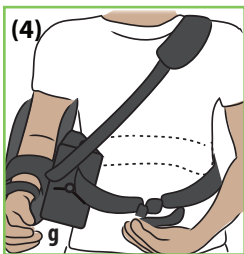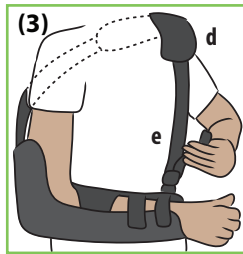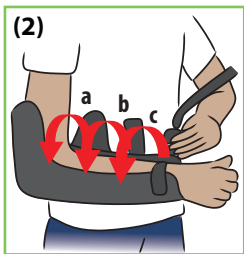

**MANAMED**  
 ManaMed, LLC  
 2612 Sirius Dr.  
 Denton, TX 76208  
 Ph: 888-508-0712  
[www.ManaMed.com](http://www.ManaMed.com)  
 Patent No. 10,123,899

©2025 ManaMed LLC. All rights reserved.  
 ManaMed, the ManaMed logo,  
 are trademarks of ManaMed LLC  
 Registered in the U.S.