

# PlasmaFlight®

Inflatable Massage System



 **MANAMED**

Recovery activated.

# PlasmaFlight®

## Indications for Use

The PlasmaFlight is intended to be an over-the-counter portable inflatable tube massage system which stimulates kneading and stroking of tissue with the hands by use of inflatable pressure cuffs.

This device can be used to:

- Temporarily increase blood circulation in the treated areas;
- Temporary relief of minor muscle aches and pains.

## Product Description

The purpose of the PlasmaFlight® is to simulate kneading and stroking of tissue with the hands by use of inflatable pressure cuffs. This is accomplished by an electronically controlled pump delivering a set amount of air to the leg cuffs that, in turn, compress the calf or calves to aid blood flow out of the lower extremities. The pump will inflate each leg cuff to a pre-set pressure of 55mmHg and deflate once the pressure is reached. The cycles are repeated on each unit until the power is turned off. Internal rechargeable batteries allow the PlasmaFlight to be completely portable, thus preventing interruptions in treatment.



## Features:

- Lightweight and Portable
- Rechargeable Battery Operated
- Easy to use

Each PlasmaFlight contains  
2 PlasmaFlight Devices, Patient Start Guide,  
Instructions for Use, AC Adapter

Product Name	Part Number
PlasmaFlight®	PFLIGHT01

2612 Sirius Drive, Denton, TX 76208

888-508-0712 (toll free)

support@ManaMed.com [www.ManaMed.com](http://www.ManaMed.com)

ManaMed and the ManaMed logo are trademarks of ManaMed LLC.



Recovery activated.