

# ManaEZ Collar 80

MD



## INSTRUCTIONS FOR USE

### DESCRIPTION OF SYMBOLS

**MD** Medical Device

### Indications for Use

The brace is intended exclusively to stabilize the cervical spine and have direct contact with the skin. The brace must be used in accordance with the physician's orders. The ManaEZ collar 80 is commonly used in the treatment of the following:

- Acute pain syndromes of the cervical spine
- Stenosis of the cervical spine
- Whiplash trauma of the cervical spine
- Fractures in the cervical spine (simple and stable)
- Dislocation & dislocation fractures of the cervical spine (postoperative)
- Traumatic injury to the cervical spine
- Radiculopathies of the cervical spine

### Product Description

The ManaEZ Collar 80 is a semi-rigid thermoplastic collar, ideal for strains and sprains of the cervical vertebrae. Two-piece design with thoracic extension. It features adjustable and lockable mandibular and occipital support. The ManaEZ Collar is lightweight, latex-free, washable and includes a breathable lining.

### Adaptation



- 1) Turn the brace lock on both sides counter-clockwise to the "unlocked" position (arrowheads in vertical position) (see figure a).
- 2) Adjust the position up or down for optimum patient support, on both sides in the corresponding position (1-9) (see figure b).  
CAUTION! Ensure that the brace setting is the same on both sides.
- 3) Turn the brace lock clockwise to the "locked" position (arrowheads in horizontal position) (see figure c).
- 4) Adjusting the inner part of the brace for small neck diameters: Hold the neck-piece of the inner part of the brace with both hands and press it together. This will allow you to bend the brace round.
- 5) Optional: Shorten the hook-and-loop closure to the desired length (see figure g).
- 6) See picture to right for back of head adjustment.

### Application

> The patient is standing or sitting down.

> Open the hook-and-loop closure on one side of the brace.

- 1) Place the inside of the brace on the neck from the front (see figure d).
- 2) Place the outside of the brace on the neck from behind (see figure e).
- 3) Fasten the hook-and-loop closure again (see figure f and g).

### Cleaning and Care


Lining can be removed and washed with a mild detergent and washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying.

### Warnings

If you experience pain, swelling, sensory changes, or discomfort with this product, please contact your doctor.

### Symbol Glossary

For an explanation of symbols, please refer to [www.manamed.com/glossary/symbol-glossary.pdf](http://www.manamed.com/glossary/symbol-glossary.pdf)

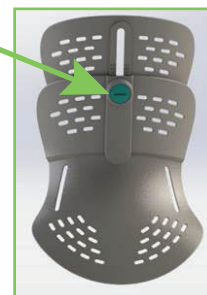

- 6) The ManaEZ Collar 80 Occipital panel was designed to allow a practitioner to have an adjustable section for the back of the head.



**MANAMED**  
ManaMed, LLC  
2612 Sirius Dr.  
Denton, TX 76208  
Ph: 888-508-0712  
[www.Manamed.com](http://www.Manamed.com)

©2024 ManaMed LLC. All rights reserved.  
ManaMed, the ManaMed logo,  
are trademarks of ManaMed LLC  
Registered in the U.S.